

# SURGICAL GUIDE BONETRUST® MINI | MINI PLUS

medical instinct®



## INSTRUMENTS:

- **Pre-Drill 1.3 mm**  
Article no.: 191-001300
- **Pre-Drill 2.0 mm**  
Article no.: 193-002000




Opening of the implant blister at the top corner by the non-sterile assistant. Let the sterile glass implant casing slide into the sterile zone.



Remove the implant from the glass casing by carefully pulling out the plastic cap.



Using the plastic cap, place the implant in the cavity and gently screw it by hand.

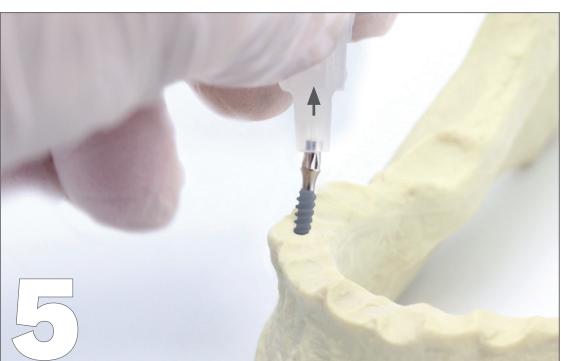

Pull the plastic cap off and screw the implant into its final position either manually or mechanically (step 6 or 7).



The following instruments are available for manual insertion.



The following instruments are available for mechanical insertion:


## INSTRUMENTS:

- **Insertion Instrument, HP**  
Article no.: 190-101250



When positioning the implant, it must be ensured that the whole rough-surfaced thread is covered by bone.

\* Bohrer unterliegen einem Verschleiss und sollten regelmäßig ausgetauscht werden.  
Eine maximale Anzahl von 20 Anwendungen sollte nicht überschritten werden.

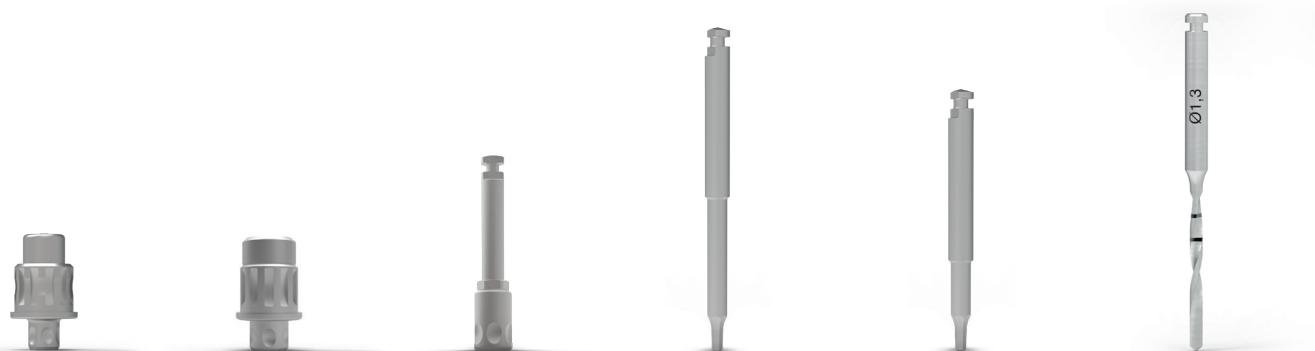

1  
Insertion Instrument  
for BoneTrust® mini  
190-101240


2  
Ratchet Adapter  
190-202001

3  
Insertion Instrument, HP  
BoneTrust® mini  
190-101250


4  
0.9 mm hex screwdriver, long  
(contra-angle insert)  
190-101213

5  
0.9 mm hex screwdriver, short  
(contra-angle insert)  
190-101209

6  
Pre-Drill 1.3 mm  
for BoneTrust® mini  
191-001300

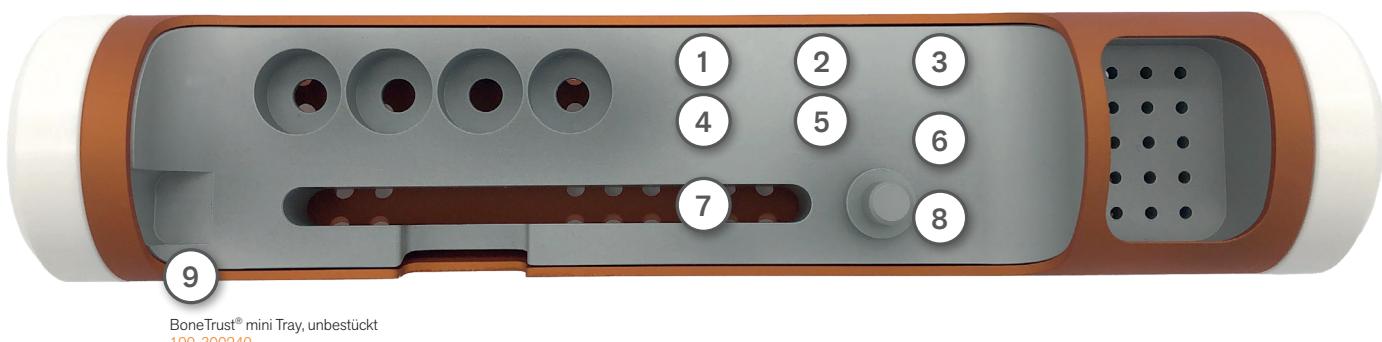

9  
BoneTrust® mini Tray, unbestückt  
190-300240

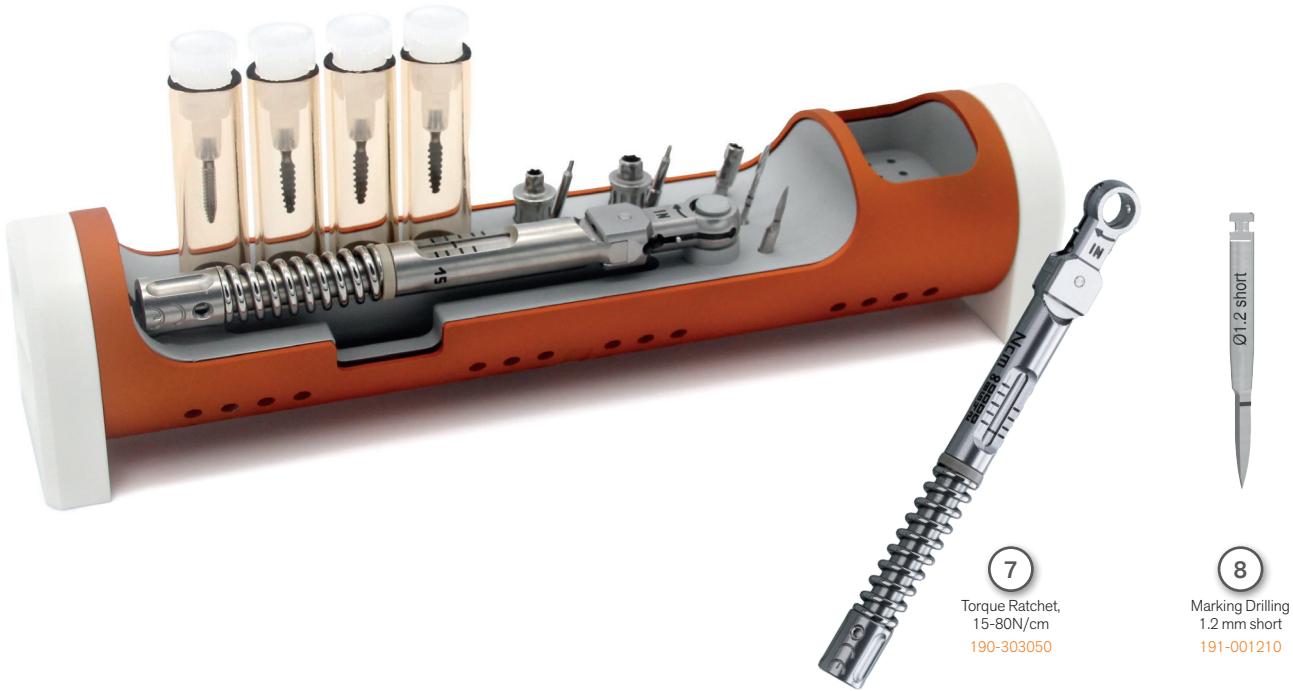

7  
Torque Ratchet,  
15-80N/cm  
190-303050

8  
Marking Drilling  
1.2 mm short  
191-001210